
## PREMIER WRIST BLOOD PRESSURE MONITOR UB-543


- 1) Install the batteries to your blood pressure monitor.  
LR03 or AAA batteries only

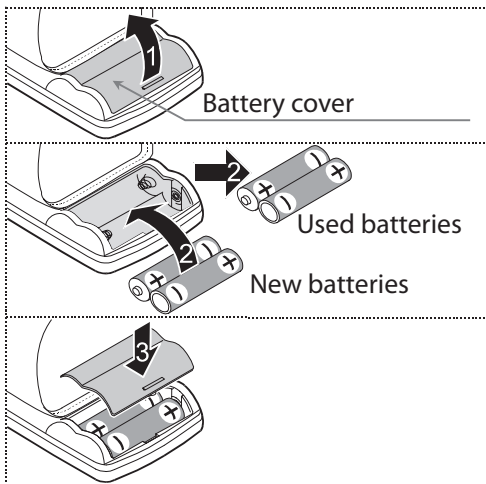- 2) (Optional) Set the Built-in Clock for date and time stamp. The operation is described in page 8 of the Instruction Manual.




3) Sit comfortably on a chair near a flat surface.

4) With palm facing upward, attach monitor to the inside of your left wrist and fasten securely. The top edge of the cuff should align with the crease of your wrist.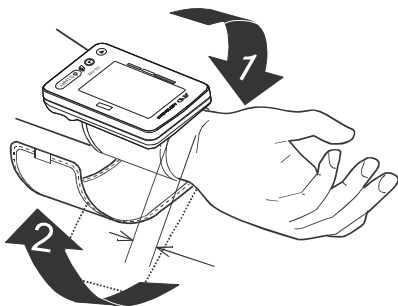


5) Press the START button.


- 6) Make sure your wrist position is at the same level with your heart. The position is critical for accurate measurement. The Correct Position Guidance (C.P.G.) feature assists to ensure the position as illustrated below.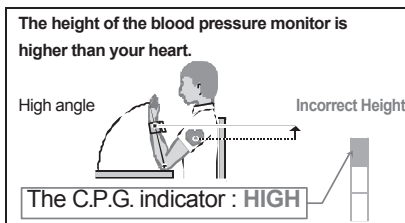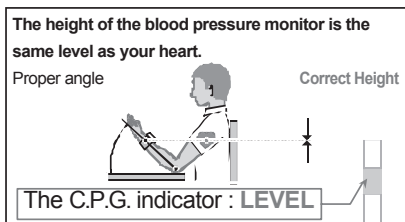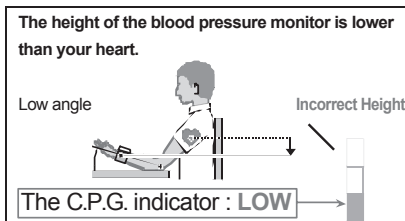


- 7) The measurement will begin 5 seconds from pressing the START button.
- 8) When the measurement is complete, the systolic and diastolic pressure and pulse rate readings are displayed. The cuff will automatically deflate completely, releasing any remaining air.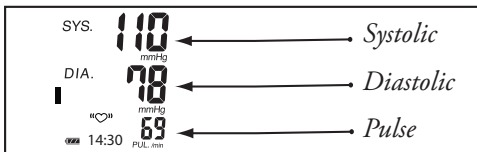

- 9) The C.P.G. will now indicate if the correct wrist position is maintained.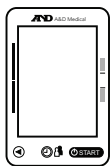

"Orange Light"  
Incorrect position

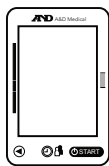"Blue Light"  
Correct position

- 10) If incorrect, please retake the measurement.

### **BLOOD PRESSURE TAKING TIPS**

- Compare blood pressure readings taken at the same time each day.
- Rest for several minutes before taking your blood pressure.
- Sit still during measurement – no talking, eating or sudden movements.

**A&D**

A&D Medical

1-888-726-9966

[www.andmedical.com](http://www.andmedical.com)

© 2014 A&D Medical. All rights reserved. Specifications subject to change without notice. UB-543-QSG-ENG-06/2014.

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